

Bone Collector Kit



HNBCK-300

1. The point shape of bone chip maker drills specially designed to collect a lot of bone.
2. The shape of bone chip maker drills and the stoppers minimize bone loss.
3. At the beginning, strongly recommends that you use 5mm stoppers.
4. Designed to fill collecting bone inside of the bone chip maker by drilling once.
5. A short drilling can prevent bone necrosis

Highlight

1. Choose appropriate bone chip maker drill and stopper in accordance with the size of bone collecting area.
2. Attach the first part of 5mm stopper to the bone chip maker drill.
3. Start drilling with 1cc of irrigation (Recommended 500RPM).
4. Make sure the status of bone collecting with the stopper fully attached.

Bone Chip Maker Drill



Drill Stopper - 10mm



Drill Stopper - 5mm

